
# EMERGO<sup>™</sup>

Lojer Merivaara multipurpose patient trolley

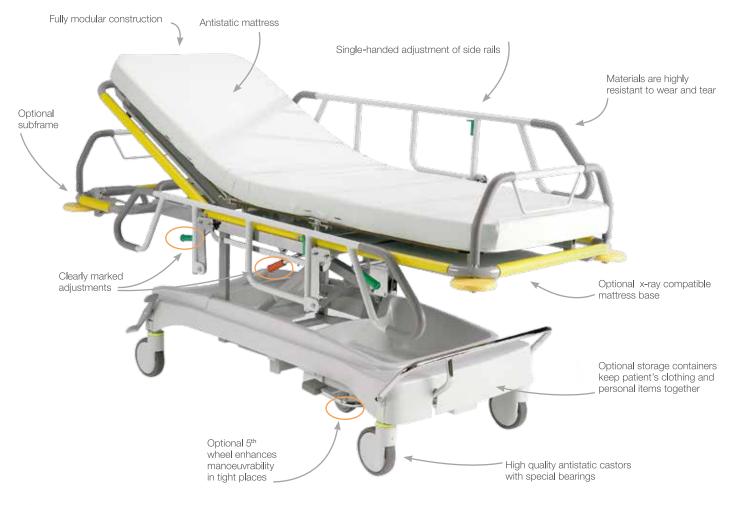

LCJER \* For easy care





#### Main features:

- 1. Modular structure that allows several different configurations
- 2. Clear and easily controllable functions
- 3. It is easy and light to carry out different care positions

The multipurpose Emergo patient trolleys (stretchers) are used for patient transportation, emergency, recovery, ICU and minor procedures. The safe and sturdy construction makes it ideal for heavy use. The versatile range of options and adjustments ensure efficient patient transfer, patient comfort and ergonomic working positions for health-care staff. The adjustable mattress base provides comfortable positions for patients to recover eliminating unnecessary patient transfer from the trolley (e.g. in sitting position).

Large, easy rolling castors provide excellent mobility and the optional fifth wheel improves overall mobility reducing the strain for the staff. Emergo has many attachment points for the IV-rod and other accessories. The wide range of accessories available will serve the demanding needs of nursing staff for many years.

#### The range:

- 6250: 2-part bed section with hydraulic height adjustment and gas spring assisted Trendelenburg and back section
- 6270: 2-part X-ray bed section with hydraulic height adjustment and gas spring assisted Trendelenburg and back section
- 6280: 2-part X-ray bed section with subframe, hydraulic height adjustment and gas spring assisted Trendelenburg and back section
- 6350: 3-part bed section with hydraulic height adjustment and gas spring assisted Trendelenburg, back section and leg section
- 6370: 3-part X-ray bed section with hydraulic height adjustment and gas spring assisted Trendelenburg, back section
- 6380: 3-part X-ray bed section with subframe, hydraulic height adjustment and gas spring assisted Trendelenburg, back section and leg section



## Factory-assembled options and accessories



Castors central locking antistatic Ø 200 mm Fixed, without castors (eg. for ships).



X-ray compatible mattress base, easy to clean. Available in models 6270, 6280 and 6370



5th wheel for improved mobility



On the left is shown how the turning radius decreases with the 5th wheel. On the right the same situation without the 5th wheel.

## Storage



ABS storage box 100006620 large 100006622 large, for dual pedals 100006625 large, for fifth wheel 100006626 small, only with large storage box (Not compatible with fixed height frame.)



Wire basket under back section



Wire basket under foot section 100000583 (Not compatible with 5th wheel.)



Accessory basket (requires trolley end) 128009784

## Accessory holders and rails



Paper roll holder 100000733



Accessory rail (pair) 700 (replaces side rail, arm rest in picture not included) 100000700 Arm rest 100000125



Oxygen bottle holder 589 100000589 for 5 kg bottles 100005860 for 2 kg bottles



Infusion rod holder 5785 100005785

### Infusion rods



Infusion rod, 4-hooks 100000499



Fixed, foldable infusion rod for subframe 100006678

# Other accessories



Hygienic and antistatic mattresses Size 650mm x 1950mm Thickness 50mm, 80mm or visco-elastic



Long push bars, detachable (pair) 6770 10000 6770 chromed, yellow padding



Patient restraining straps (set) 100000790




Trolley End 567 100000567



Side rails (pair) 100006713 chromed/yellow



Push bar, foldable 100006793 chromed/yellow 100006795 epoxy coated/yellow padding

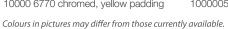



Technical data and fetures						
Model	6250	6270	6280	6350	6370	6380
Intended use	Emergency, transport, minor procedures	Emergency, ICU, minor procedures	Emergency, ICU, minor procedures	Emergency, transport, minor procedures, recovery	Emergency ICU, transport, minor proce- dures, recovery	Emergency ICU, trans- port, minor procedures, recovery
Base sections	2	2	2	3	3	3
X-ray platform	No	Yes	Yes	No	Yes	Yes
Subframe for equipment	No	No	Yes	No	No	Yes
Width	750 mm	750 mm	750 mm	750 mm	750 mm	750 mm
Patient surface width	705 mm	705 mm	705 mm	705 mm	705 mm	705 mm
Length	2075 mm	2075 mm	2150 mm	2075 mm	2075 mm	2150 mm
Recommended mattress size	650 x 1950 mm	650 x 1950 mm	650 x 1950 mm	650 x 1950 mm	650 x 1950 mm	650 x 1950 mm
Height adjustment	560 - 930 mm	625 - 995 mm	625 - 995 mm	560 - 930 mm	625 - 995 mm	625 - 995 mm
Safe Working Load (SWL)	225 kg	225 kg	225 kg	225 kg	225 kg	225 kg
Trendelenburg	14°	14°	14°	14°	14°	14°
Anti-Trendelenburg	7°	7°	7°	7°	7°	7°
Back section adjustment	0 - 70°	0 - 70°	0 - 70°	0 - 70°	0 - 70°	0 - 70°
Leg section adjustment	-	-	-	0 - 45°	0 - 45°	0 - 45°
Castors	Central locking, antistatic 150 mm	Central locking, antistatic 150 mm	Central locking, antistatic 150 mm	Central locking, antistatic 150 mm	Central locking, antistatic 150 mm	Central locking, antistatic 150 mm
Country of manufacture	Finland	Finland	Finland	Finland	Finland	Finland
CE marked	Yes	Yes	Yes	Yes	Yes	Yes

#### Other features:

- Yellow bumpers in each corner
- Pedals for hydraulic height adjustment on both sides
- Trendelenburg and anti-Trendelenburg adjustments
- Antistatic central locking 150 mm castors
- Gas-spring assisted back section
- 2- or 3-part base











© Lojer Group 2020. All Rights reserved | Lojer Group, Putajantie 42, 38210 Sastamala | Tel. + 358 (0) 10 830 6700 | info@lojer.com | www.lojer.com

The story of Lojer Group dates back to 1919 when Vammalan Konepaja was founded. In the early years, the company repaired f.ex. bicycles. Today, the company is the largest manufacturer of medical furniture in the Nordic countries and our products are used in 115 countries.